






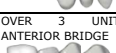

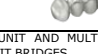



How to Select a Material!

1. In the first column chose the indication you are planning to restore 2. Follow that row to the right to see which materials are suitable for your indication and what type of cementation you should use. 3. Follow the column down to see important informations about the material choice you made.

INDICATION	Bio Premium ZR	BIO ZIR	BIO FZ PLUS	IPS e.max	ZOLID/CUBEX	BIO MULTI FZ	CEMENTATION
THIN VENEERS 0.3 mm 	NO	NO	NO	YES ADHESIVE CEMENTATION	NO	NO	ADHESIVE MULTILINK AUTOMIX
VENEERS 	NO	NO	NO	YES ADHESIVE CEMENTATION	NO	NO	ADHESIVE MULTILINK AUTOMIX
OCCLUSAL VENEERS 	NO	NO	NO	YES ADHESIVE CEMENTATION	NO	NO	ADHESIVE MULTILINK AUTOMIX
INLAYS ONLAYS 	NO	NO	NO	YES ADHESIVE CEMENTATION	NO	POSSIBLE BUT NOT THE BEST	ADHESIVE MULTILINK AUTOMIX
ANTERIOR CROWN 	YES	YES	YES	YES ADHESIVE CEMENTATION	YES	NO	ADHESIVE OR SELF ADHESIVE OR CONVENTIONAL
POSTERIOR CROWN 	YES	YES	YES	YES ADHESIVE CEMENTATION	YES	YES	ADHESIVE OR SELF ADHESIVE OR CONVENTIONAL
3 UNIT ANTERIOR BRIDGE 	YES	YES	YES	YES ADHESIVE CEMENTATION	YES	NO	ADHESIVE OR SELF ADHESIVE OR CONVENTIONAL
OVER 3 UNIT ANTERIOR BRIDGE 	YES	YES	YES	YES ADHESIVE CEMENTATION	NO	NO	ADHESIVE OR SELF ADHESIVE OR CONVENTIONAL
3 UNIT POSTERIOR BRIDGE 	YES	YES	YES	YES UP TO THE SECOND PREM. ADHESIVE CEMENTATION	NO	YES	ADHESIVE OR SELF ADHESIVE OR CONVENTIONAL
4 UNIT AND MULTI UNIT BRIDGES 	YES	YES	YES	NO	NO	YES	ADHESIVE OR SELF ADHESIVE OR CONVENTIONAL
SCREW RETAINED CROWNS 	YES	YES	YES	YES	YES	YES	N/A
PRICE RANGE	\$\$\$\$	\$\$\$	\$\$\$	\$\$\$	\$\$	\$	
AESTHETIC RATING	★★★★+	★★★★	★★★	★★★	★★★	★★	
SHADE GUIDES	Vita Classical+ 3D Master	Vita Classical+ 3D Master	Vita Classical+ 3D Master	Vita Classical Only	Vita Classical Only	Vita Classical Only	
WARRANTY	1 YEAR	1 YEAR	1 YEAR	Veneers NO WARRANTY, Crowns 5 YR	5 YEARS	10 YEARS	
CONTRAINDICATIONS	Parafunctions, e.g. bruxism	Parafunctions, e.g. bruxism	Generally not Indicated for the mandibular arch	Parafunctions, e.g. bruxism Provisional insertion/trial wear period	CANTILVERS Parafunction, e.g. bruxism	Aesthetic Zone	
OTHER IMPORTANT FACTORS TO CONSIDER	Dark Abutments e.g. Metal Post, Ti Abutment can drastically affect the Shade! Please advise the Lab!!	Dark Abutments e.g. Metal Post, Ti Abutment can drastically affect the Shade! Please advise the Lab!!	Dark Abutments e.g. Metal Post, Ti Abutment can drastically affect the Shade! Please advise the Lab!!	Dark Abutments e.g. Metal Post, Ti Abutment can drastically affect the Shade! Please advise the Lab!!	Dark Abutments e.g. Metal Post, Ti Abutment can drastically affect the Shade! Please advise the Lab!!	Dark Abutments e.g. Metal Post, Ti Abutment can drastically affect the Shade! Please advise the Lab!!	
TOOTH REDUCTION REQUIREMENTS	1.5 mm Feather Edge preparation is possible but not ideal!	1.5 mm Feather Edge preparation is possible but not ideal!	1.5 mm Feather Edge preparation is possible but not ideal!	Shoulder Preparation Anteriors Minimum 1.2 mm Posteriors Min. 1.5m	Ideal 1.5mm Minimum Occlusal: Anteriors 1mm Posteriors Min. 1.2 mm Feather Edge preparation is possible but no warranty!	Ideal 1 mm Minimum Occlusal : 1mm Circumf. Min. 1mm	